Instructions For Use - HyFlex® NT NiTi

Root Canal Instruments

Indication for Use: For dentin removal and root canal shaping.

Contraindications: None known.

 WARNING!	Files are provided non-sterile, and must be sterilized before use.
	This product consists of a Nickel Titanium alloy and chromium. DO NOT USE for individuals with known allergic sensitivity to these metals.
	This product contains a chemical known to the state of California to cause cancer.

Instructions:

A new file should be used for each procedure to reduce the risk of file separation.

- Sterilize files prior to use. See sterilization instructions below.
- Use of a slow speed hand piece required. Operate the hand piece at 300 RPM (revolutions per minute). Recommended torque setting up to 2.5 N·cm (25 mN·m).
- Irrigate during and between each step. COLTENE® ENDO recommends the use of CanalPro™ EXTRA for irrigation and debridement.
- Clean file flutes after each insertion into the root canal.
- Recapitulate between each step.

Files may be used with Crown-down and Step-back techniques, or with the Hybrid technique below. In curved canals, use of a HyFlex® CM™ (Controlled Memory) is recommended for preparing the apical curvature.

Step by Step instructions:

Coronal Patency: After gaining a Straight-Line coronal access, create an apical glide path using HyFlex® GPF™ system or a 02/20 hand file size.

STEP 1 - Start with the 08/25 file. Use as an orifice opener. Move slowly forward without pressure in a pecking motion. Once resistance occurs go to step 2. Check patency with 02/20 hand file, irrigate and lubricate. Optional Step: If working length is reached in large canals it may be possible to go directly to step 4.

STEP 2 - Use the 04/20 file for apical enlargement to the Working Length. Move slowly forward without pressure in a pecking motion. In case resistance occurs use a 02/20 hand file to check patency. Irrigate and lubricate.

STEP 3 - Insert 04/25 file for apical enlargement to the Working Length. Move slowly forward without pressure in a pecking motion. If resistance occurs, go back to the previous step. Use a 02/20 hand file to check patency. Irrigate and lubricate.

STEP 4 - Use the 06/20 file to shape middle part of the root canal. This file can also be taken to Working Length. Move slowly forward without pressure in a pecking motion. If resistance occurs, go back to the previous step. Use a 02/20 hand file to check patency. Irrigate and lubricate.

STEP 5 - Take 04/30 file to Working Length to enlarge the apical part of the canal. Move slowly forward without pressure in a pecking motion. If resistance occurs, go back to the previous step. Use a 02/20 hand file to check patency. Irrigate and lubricate.

STEP 6 - Finish treatment with 04/40 file to Working Length. Move slowly forward without pressure in a pecking motion. If resistance occurs, go back to the previous step. Use a 02/20 hand file to check patency. Irrigate and lubricate.

Cleaning Instructions:

- Soak all instruments immediately after use in a detergent and disinfecting solution combined with proteolytic enzyme if possible.
- In case of having debris left on the file, use a soft toothbrush for cleaning.
- Follow instructions below under "Sterilization Instructions".

Coltene/Whaledent Inc. 
235 Ascot Parkway
Cuyahoga Falls, Oh 44223/USA
Tel. USA/Canada 1 800 221-3046
+1 330 916-8800
Fax +1 330 645 8704
info.us@coltene.com

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

 www.coltene.com

Upon request a hardcopy of the IFU will be supplied free of charge within 7 days.

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- Remove new or un-used files from the packaging prior to sterilization. Note that due to the unique material features of the file, they may be slightly bent during packaging process. This can be corrected by hand.
- Place the devices in a file block, support or container to avoid any contact between instruments.
- If a disinfecting solution contains a corrosion inhibitor, it is recommended to rinse the instruments just before the autoclaving. Do not use bleach.
- Package the kit in a sterilization wrap. Steam sterilization at: 132°C / 270°F for 4 minutes, or 18 min for prion inactivation with a 20 minute dry time.
- Keep devices in sterilization packaging in a dry and clean environment

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Sterilization Instructions: (LOCATION OF COLOR BAND DETERMINES THE PROCEDURE)

Sterilization Instructions: (LOCATION OF COLOR BAND DETERMINES THE PROCEDURE)

- Remove files from the packaging prior to sterilization. Note that due to the unique material features of the file, they may be slightly bent during packaging process. This can be corrected by hand.
- Place the devices in a file block, support or container to avoid any contact between instruments.
- If a disinfecting solution contains a corrosion inhibitor, it is recommended to rinse the instruments just before the autoclaving.
- Put the kit in sterilization pouches. Steam sterilization at: 134°C / 273°F for 6 minutes, or 18 min for prion inactivation with a 20 minute dry time.
- Keep devices in sterilization packaging in a dry and clean environment

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